

READY HEAT

INFANT WARMING MATTRESS

UP TO 8 HOURS OF HEAT

WARMS TO 100°F (38°C)



INSTRUCTIONS FOR USE

1. DO NOT open the outer protective package until ready to use.
2. Remove the mattress from the outer protective package and expose the Ready-Heat Infant Mattress to air. DO NOT open mattress pocket or inside pouch.
3. Mattress will start to heat immediately. Allow 10-15 minutes to reach operating temperature.
4. Place infant on mattress with the Blue side Up, towards the infant and the Beige side Down, away from the infant.
5. If heat decreases, expose mattress to air.
6. After use, dispose of mattress with regular waste. All natural ingredients; will not harm environment.



WARNINGS **Failure to follow these precautions may result in serious burns or other serious injuries.**

1. **DO NOT use mattress on exposed skin.**
2. Careful supervision is required for use with the elderly, infants, children, people with sensitive skin and those who can't feel heat or regulate heat. Keep out of reach of children.
3. People with allergies, diabetes, frostbite, scars, open wounds or circulatory problems should consult a physician before using the Ready-Heat Infant Mattress
4. Remove immediately if mattress becomes too hot or if the infant is uncomfortable. Maximum temperature may reach 108°F (42°C). DO NOT use while sleeping.
5. DO NOT open the sealed pockets or pouch and DO NOT allow the contents to contact mouth or eyes. If contact with skin or eyes, wash with clean water. Contents contain iron powder which, if swallowed, can be harmful. If contents are swallowed, immediately call a poison control center.
6. DO NOT fold the mattress as folding may increase heat.
7. DO NOT use in oxygen rich environment, including hyperbaric chamber or oxygen tent.

CONTENTS: Iron powder, water, activated carbon, wood powder, salt and polyethylene foam

We recommend the Ready-Heat Infant Warming Cocoon be used in conjunction with our Ready-Heat Infant Warming Mattress, the Mattress to be placed outside & underneath for constant warming should the Cocoon need to be opened for patient access.



TechTrade



TechTrade, LLC • 6900 Tavistock Lakes Blvd. • Suite 400 • Orlando, FL 32827 • USA
1-866-491-TECH • Fax: 201-706-8131 • info@techtradellc.com • www.ready-heat.com

SIZE: 11" x 13" (28 cm x 33 cm)
PART# S2RHIM
WEIGHT: 7 oz. (198 gm)



EMERGO EUROPE
Westervoortdijk 60,
6827 AT Arnhem
The Netherlands



Please visit www.ready-heat.com/IFU
for complete IFU and Symbol Descriptions.

MADE IN USA

0459



CLEARED



(01)00850017905078