

READY HEAT

FULL BODY TEMPERATURE MANAGEMENT BLANKET

UP TO 10 HOURS OF HEAT



INSTRUCTIONS FOR USE

1. This blanket provides warm relief in Pre-/Intra-/Post-Operation; aids in the prevention of hypothermia.
2. DO NOT open the outer protective package until ready to use, as activation will start immediately.
3. When ready to use, remove the blanket from the outer protective package (keep outer protective package for later use).
4. Unfold the Ready-Heat Temperature Management Blanket to expose to air – heat activation will start automatically.
5. If the patient is not wearing a gown, place a drape over his/her body. DO NOT PLACE THE BLANKET DIRECTLY ON EXPOSED SKIN.
6. Place the blanket on the patient as indicated by the sticker, with the sticker face-up and on the patients left side and the arrows pointing toward the patients head.
7. The blanket will start warming within 3 minutes. Allow 20-25 minutes for the blanket to reach full operating temperature
8. FOR PROPER DISPOSAL: After use fold the blanket, place back in the outer protective package and dispose with regular waste. If the blanket was in contact with blood-borne pathogens or other bodily fluids, place back in the outer protective package and dispose with medical waste.



WARNINGS **Failure to follow these precautions may result in serious burns or other serious injuries.**

1. This blanket is not sterile and is intended for single use only. DO NOT re-use.
2. Careful supervision is required for use with the elderly, people with sensitive skin and those who can't feel heat or regulate heat. Keep out of reach of children.
3. **DO NOT PLACE THE BLANKET DIRECTLY ON EXPOSED SKIN.**
4. Monitor the temperature and cutaneous response of patients who are incapable of reacting, communicating and/or who are without a sense of feeling every 10-20 minutes or according to institutional protocol.
5. Continually monitor vital signs for all patients on a regular basis. DO NOT leave patients unmonitored during any duration of use.
6. Immediately notify physician of any vital sign instability.
7. DO NOT fold blanket while on the patient. Slide or remove blanket to access patient. Folding may cause heat increase.
8. Remove blanket immediately if it becomes too hot or if the patient becomes too warm. Maximum temperature may reach 43°C.
9. DO NOT open the sealed pockets. DO NOT allow the contents of the sealed pockets to contact mouth or eyes. If contact occurs, wash area with clean water. If contents are swallowed, call poison control center immediately.
10. DO NOT use this blanket under the patient.
11. DO NOT place instruments on the blanket at any time.
12. DO NOT use on ischemic limbs; physician must determine if proper to use with patients with diabetes, frostbite, scars, open wounds or circulatory problems.
13. DO NOT use in oxygen rich environment, including hyperbaric chamber or oxygen tent.

CONTENTS: Iron powder, water, activated carbon, wood powder and salt



TechTrade



TechTrade, LLC ▪ 6900 Tavistock Lakes Blvd. ▪ Suite 400 ▪ Orlando, FL 32827 ▪ USA
1-866-491-TECH ▪ Fax: 201-706-8131 ▪ info@techtradellc.com ▪ www.ready-heat.com

SIZE: 34" x 60" (86 x 152 cm)
PART# SB9RH9120
WEIGHT: 18 oz. (510 gm)



EMERGO EUROPE
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands



MADE IN USA



0459



CLEARED



(01)00850017905054

Please visit www.ready-heat.com/IFU
for complete IFU and Symbol Descriptions.

RHPLG-8 rev 8 - JAN 2026